DE / EN

Gebrauchsanweisung Art.-Nr. 101890

**schwa-medico**  
MENSCHLICHE MEDIZIN

# Phalli Max

ELEKTROSTIMULATIONSGERÄT

**Inhalt**

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ELECTROSTIMULATION DEVICE

## Contents

|      |   |    |     |   |    |
|------|---|----|-----|---|----|
| 1    | General Information                           | 23 | 4   | Electrodes  | 32 |
| 1.1  | Intended purpose                              | 23 | 4.1 | Self-adhesive electrodes                            | 32 |
| 1.2  | Instructions regarding safe use of the device | 23 | 4.2 | Loop electrodes                                     | 33 |
| 1.3  | Contraindications                             | 24 | 5   | Programs  | 34 |
| 1.4  | What side effects may occur?                  | 24 | 5.1 | Program 1   | 34 |
| 2    | Description of the PhalliMax                  | 25 | 5.2 | Program 2   | 35 |
| 2.1  | Description of the device                     | 25 | 5.3 | Program 3   | 35 |
| 2.2  | Symbols                                       | 26 | 6   | General Information                                 | 36 |
| 2.3  | Technical data                                | 26 | 6.1 | Warranty  | 36 |
| 2.4  | Classification                                | 27 | 6.2 | Guarantee   | 36 |
| 2.5  | Pulse type                                    | 27 | 6.3 | Care and cleaning                                   | 36 |
| 3    | Operating the PhalliMax                       | 28 | 6.4 | New settings, changes and repairs                   | 36 |
| 3.1  | Starting and charging the integrated battery  | 28 | 6.5 | Safety checks (§11 MPBetreibV)                      | 36 |
| 3.2  | Connecting the cable and the electrodes       | 28 | 6.6 | Combination with original accessories               | 37 |
| 3.3  | Turning on the device                         | 29 | 6.7 | Reference sheet for battery retraction and disposal | 37 |
| 3.4  | Selecting the therapy program                 | 29 | 7   | Scope of delivery/Accessories                       | 38 |
| 3.5  | Starting the simulation                       | 29 | 7.1 | Scope of delivery                                   | 38 |
| 3.6  | Pausing the stimulation                       | 30 | 7.1 | Optional accessories                                | 38 |
| 3.7  | Stopping the stimulation                      | 30 |     |   |    |
| 3.8  | Setting the program time                      | 30 |     |   |    |
| 3.9  | Locking the device                            | 31 |     |   |    |
| 3.10 | Muting the audio signal                       | 31 |     |   |    |
| 3.11 | Retrieving data from the therapy memory       | 31 |     |   |    |
| 3.12 | Turning off the device                        | 32 |     |   |    |

## 1 General Information

### 1.1 *Intended purpose*

The PhalliMax serves the treatment of erectile dysfunction caused by either weakness of, or damage to, the muscle cells and muscles in the cavernous body system and/or in the pelvic floor region.

Health disorders may be a sign of serious illnesses. Please consult your physician!

### 1.2 *Instructions regarding safe use of the device*

Please read the instructions for use carefully prior to using the device!

- » The PhalliMax can only be used with its original accessories.
- » Keep water and other liquids away from the PhalliMax.
- » Never use the PhalliMax if it is malfunctioning or damaged.
- » The PhalliMax should not be used at the same time as high-frequency devices, such as cellphones, because interactions may occur between the electromagnetic field of the device and the PhalliMax resulting in increased power discharge.
- » Current density of over 2 mA/cm<sup>2</sup> on all electrode surfaces may require greater attention on the part of the operator.
- » The minimum electrode size should not be less than 2 cm<sup>2</sup>.
- » Each device must only be connected to a single patient.
- » Electrotherapy should not be applied above or through the head, directly on the eyes, under the covering of the mouth, on the front of the neck (especially the carotid sinus), or with electrodes attached on the chest and upper back or across the heart.
- » The device must not be used while operating machinery or during activities that require greater attention. This particularly applies to street traffic.
- » Do not drop the PhalliMax device or handle it improperly, expose it to extremes of temperature or high air humidity (only use at temperatures between 5 °C – 40 °C and at relative humidity less than 80%).
- » Close supervision is necessary if the PhalliMax is operated in the vicinity of children.
- » Store the PhalliMax in its original packaging to protect it from damage and contamination.

### 1.3 Contraindications

The PhalliMax must not be used in (or may be used only after consulting a physician)

- » Patients with electronic implants such as heart pacemakers, defibrillators or pumps
- » Patients with cardiac rhythm disorders
- » Patients subject to seizures (epilepsy)
- » Patients with skin disorders in the vicinity of the electrodes
- » Patients with malignant diseases in the area of application
- » Patients with sensitivity disorders or skin sensitivity in the area of application.

### 1.4 What side effects may occur?

#### Skin intolerances

- » Skin intolerances may arise as a reaction to the electrodes, the electrode gel, or the electrical impulses themselves. In the event of persistent redness, burning sensation, itching, blisters, or wounds under the electrodes or in the area surrounding the area that they are applied, which occur after stimulation, a physician must be consulted before any further application. Mild, non-persistent redness of the skin following stimulation near the electrodes is normal, because the flow of blood is improved as a result of the stimulation.

#### Muscle overexertion (“sore muscles”)

- » As with any other type of muscle training, muscle overexertion can also occur in electrotherapy. In the event of muscle overexertion following stimulation of the pelvic floor muscles, the duration and frequency of application of the PhalliMax should be reduced.

#### Temporary pain

- » Despite all precautions, temporary pain may occur in the area of stimulation during or after stimulation. Most of the time a slight repositioning of the electrodes or a shorter application time will help. The pain often subsides quickly and usually only occurs at the start of electrotherapy. If the pain persists, seek medical advice before any further applications of the PhalliMax.

## 2 Description of the PhalliMax

### 2.1 Description of the device

The PhalliMax was designed for nerve and muscle stimulation. The individual programs can be selected using the buttons to make the respective settings. The various operating states are shown on the display.

- 1 Display
- 2 Program selection
- 3 Editing button for modifying the settings
- 4 Modification buttons for intensity of channels 1 and 2  
These buttons are used to change the different parameters in editing mode
- 5 On/Off button
- 6 Cable output connectors for the electrodes
- 7 Charging cable connection
- 8 ON/OFF sliding switch (ON/OFF)

## 2.2 Symbols

Serial number

Catalogue number

Follow instructions for use

Attention: Read accompanying documents, especially user manual!

Type BF applied part

Manufacturer

Date of manufacture

Environment

Do not dispose of the appliance with the normal household waste at the end of its life, but hand it in at an official collection point for recycling. By doing this you will help to protect the environment.

☞0482 By labelling with CE mark, the manufacturer asserts that the product fulfills all relevant requirements of the respective EC Directives. A conformation assessment process has been successfully completed. The identification number in respect of the conformation assessment process is given in accordance with the CE labelling of the relevant notified body.

## 2.3 Technical data

2-channel nerve stimulator with constant voltage characteristic, contact control, output circuit breaker protection and 3 integrated programs.

|   |                                       |
|---|---------------------------------------|
| Operating voltage   | 4.8 volt                              |
| Power supply  | 4.8 volt NIMH battery pack            |
| Current consumption for minimum load                          | approx. 3 mA                          |
| Current consumption for maximum load                          | approx. 130 mA                        |
| Max. output current at load of $R_L = 1 \text{ k}\Omega$ real | 100 mA, adjustable in 0.5 mA steps    |
| Pulse type  | positive rectangle with negative part |
| Pulse width   | 50–400 $\mu\text{s}$                  |
| Frequency / Frequency range                                   | 1–120 Hz                              |

|  |                        |
|--|------------------------|
| Timer  | 1–99 min               |
| Switching the operating display to “Low-Batt.” | 4.4 V                  |
| Response of contact control at output current  | 10 mA                  |
| Dimensions (H x W x D)                         | 110 mm x 70 mm x 25 mm |
| Device weight (including battery)              | approx. 150 g          |


Note: The respective tolerance is  $\pm 15\%$  if not indicated otherwise.

## 2.4 Classification


The PhalliMax is categorized as Class IIa according to Annex IX (Rule 9) of the EC Directive on Medical Products.

## 2.5 Pulse type


*At load of 1k $\Omega$  real*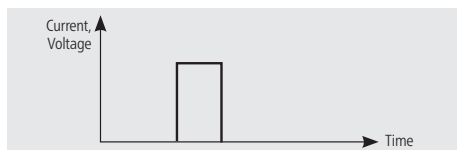

*Change in the output current in relation to load resistance*

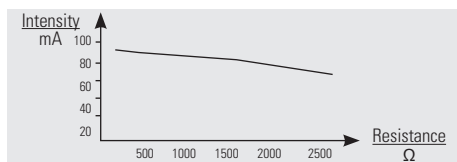*At load of ANSI/AAMI standard*
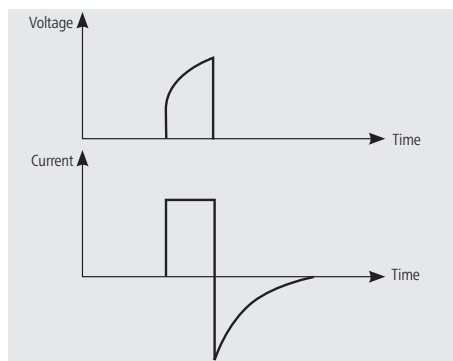
## 3 Operating the PhalliMax

### 3.1 Starting and charging the integrated battery

Please fully charge the integrated battery before the first use of the PhalliMax. To do so, proceed as follows:

- » Set the sliding switch on the bottom of the device to OFF.
- » Plug the charger cable into the charging cable connection between the two outputs for the electrodes.
- » Then connect the charger to a socket.
- » The pilot lamp on the charger's plug will light up red to indicate that the battery is being charged.
- » When charging has been completed, the pilot lamp will change from red to green.
- » Now pull the charger's plug out of the socket and remove the charger cable from the PhalliMax.

**Important!** Please move the sliding switch on the bottom of the device back to the ON position after charging the battery. Otherwise it is not possible to turn on the PhalliMax.

**Note:** The battery's charging status is indicated as a battery of four segments in the upper right corner of the display. If the voltage drops below 4.4 V, the battery symbol will start flashing. In such cases, please charge the battery in order to be able to continue using the PhalliMax. If the voltage falls to a critical level so that only one segment or less is displayed, then the device will switch off and cannot be restarted until you recharge it.

### 3.2 Connecting the cable and the electrodes

The PhalliMax has two channels. It is possible to just use one of the channels or to use both in parallel. You can set the intensity for each channel separately. However, for the electrode positioning recommended for treating erectile dysfunction, you will only need one channel. Please ensure that during treatment you set the intensity only for the channel to which you have connected the electrode cable.

Attach two electrodes to the two electrode connectors on the cable. Connect the other end of the cable to one of the two outputs on the bottom of the PhalliMax. Then place the electrodes on your skin.

You can now turn on the PhalliMax.

**Important!** Do not affix the electrodes to irritated or diseased areas of skin. Please turn off the device before changing the position of one or more of the electrodes.

### 3.3 Turning on the device

The sliding switch must be in the ON position so that you can turn on the PhalliMax. Turn on the device using the -switch. A brief audio signal will indicate that the PhalliMax is ready.


The display now shows the most recently used program, frequency and pulse width, therapy time, intensity and battery charging status. If the locking feature has been activated, a lock icon will appear next to the program number.

### 3.4 Selecting the therapy program

The PhalliMax starts by using the therapy program that was last used. To change to the next program, please press the -button. If the device is set on Program 3 (P-03), it changes back to Program 1 the next time you press the -button. Alternatively, you can use the left -button to switch to the previous program.

**Note:** If the device is locked, it is not possible to change the set program. Read the section “Locking the device” to find out more about locking and unlocking the PhalliMax.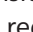

### 3.5 Starting the simulation

Make sure that you have selected the right program and that the electrodes and cables have been connected correctly. Start the stimulation with the -button of the channel you connected the cable to. The intensity increases by 0.5 mA with each press of the -button. If you hold down the -button, the intensity will be increased until you release the button. You can use the -button to reduce the intensity in the same manner.

The currently selected intensity is shown on the display. It can be set within a range of 0.5–100 mA.

### Safe control of the intensity

To avoid any accidental increases in intensity, the intensity is locked by the PhalliMax at the level it was last set at if it is not operated for 10 seconds. The display

then shows a key icon next to the program number. To increase the intensity again, it has to be lowered by at least one step (0.5 mA) using the -button. Then you can increase the intensity to the desired level using the .

If the electrodes or the cable are not adequately or correctly attached, the intensity of the PhalliMax cannot be set higher than 10 mA, and it will automatically be reset to zero.

### 3.6 Pausing the stimulation

If you want to pause the stimulation, please press the -button. Instead of the time, the word PAUS will now appear on the display, and the stimulation will be stopped. Pressing the -button again restarts the stimulation at the level of intensity that was last selected.

### 3.7 Stopping the stimulation

Stimulation can be stopped at any time by holding down the -button or the -button for one second or by reducing the intensity to zero. The stimulation stops automatically once the total program time has been completed for the selected program.

### 3.8 Setting the program time

The runtime for all programs is preset to 30 minutes on all of the PhalliMax programs. To change the program runtime for the set program, press the -button when the device is turned on. The time will start flashing and can then be changed within a range of 1–99 minutes by pressing the right or left -button. Press the -button again to have the program runtime stored for that program.

You can also use the PhalliMax without a set program runtime. To turn off the timer, press the left -button and the -button at the same time for 3 seconds. Now you will see TIMER and the runtime on the display. Please press . Instead of the program runtime, the display will show -- --. Press the -button to store the timer settings. You can do the same thing to turn the timer back on.

If you have turned off the timer and are using the PhalliMax without any set program runtime, the stimulation will not stop after any predetermined time period. It will run until it is manually stopped.

### 3.9 Locking the device

To prevent the set program from being changed, the program selection can be locked. To do so, press the right -button and the -button at the same time for 3 seconds. A key icon will appear next to the program name. Then you can only perform therapy with the set program and you will be unable to make any other settings.

Press the right -button and the -button at the same time for 3 seconds again to unlock the device.

### 3.10 Muting the audio signal

As soon as you press a button on the PhalliMax you will hear a brief audio signal. To mute the audio signal, press the left -button and the -button at the same time for 3 seconds. You will then see BEEP and ON or OFF. Press  to switch the sound on or off. Then press the -button to save the settings.

### 3.11 Retrieving data from the therapy memory

The PhalliMax has an internal memory that is used to store the data from the completed therapy sessions. You can retrieve the data from the last 30 days of therapy whereby up to 3 therapy sessions per day are stored in memory.

Press the right -button and the -button at the same time for 3 seconds to open the therapy memory. If the therapy memory is empty, the word "NULL" ["NONE"] will appear on the display for two seconds. If the therapy memory already contains data, you can view the following data:

You can use the left -button to retrieve the values stored for the respective previous therapy day, while the left -button can be used to view the values from the next therapy day. You can scroll through the data from the individual therapy sessions from one day using the right -buttons. You can use the right -button to get to the previous therapy session, while the left -button can be used to switch to the next therapy session.

When you press the **P**-button, instead of the number of the therapy day and the therapy session, the average intensity will be displayed that was used for stimulation during the respective session. You will see the intensity for channel 1 on the left side, and the intensity for channel 2 will be displayed on the right.

**Note:** The therapy days in the therapy memory refer to the days on which stimulation took place using the PhalliMax. They have nothing to do with the calendar date and do not account for any of the days on which the PhalliMax was not used. For example: The first therapy was performed on Monday, the second on Tuesday, and the third on Friday. In this case Monday is therapy day 1, Tuesday is therapy day 2, Tuesday, and Friday is therapy day 3.

Holding down the **E**-button for 3 seconds deletes the therapy memory. In that event the display will show "Clr" for two seconds. You will then automatically leave the therapy memory display.

### 3.12 Turning off the device

While stimulation is off, press the **⏻**-button to turn off the PhalliMax. If the charge on the battery is no longer sufficient, or if the device is not operated for more than 2 minutes without running stimulation, it will turn off automatically.

## 4 Electrodes

You can use the PhalliMax either with self-adhesive electrodes or with a loop electrode and a self-adhesive electrode. The following instructions are intended to help you use the electrodes correctly.

### 4.1 Self-adhesive electrodes

Self-adhesive electrodes are reusable. To increase their service life, please note the following recommendations:

- » Clean your skin with water and soap before attaching the electrodes. It should be dry and as free of oil as possible.
- » Do not use ointments, creams, etc. before attaching the electrodes
- » Do not affix the electrodes to damaged or diseased areas of skin.

- » To remove an electrode from the carrier foil that they are on inside of the packaging, hold one end of the electrode and gently pull it off of the foil. Please never tug at the cable, because that might damage the electrodes.
- » Remove the electrodes from the skin in the same manner once therapy has been completed, without tugging at the cable.
- » To release the cable between the PhalliMax and the electrode from the electrode, hold both plug connectors and pull them apart. Never tug at the cable itself.
- » Stick the electrodes back onto the carrier foil after each use and put them back into the polyethylene bag.
- » To ensure that the electrodes last as long as possible, store them in a cool place, e.g. in the refrigerator.
- » Do not use the electrodes in temperatures below + 5 °C or over + 35 °C.
- » Any gel residue can be easily removed from the skin using water and soap.

## 4.2 Loop electrodes

The loop electrode is made of conductive silicon rubber and is reusable. In the treatment of erectile dysfunction, it is placed around the penis and connected to the PhalliMax by the cable. When using the loop electrode, you will always need a self-adhesive electrode to serve as a opposite-electrode.

- » Clean your skin with water and soap before attaching the electrodes. It should be dry and as free of oil as possible.
- » Do not affix the electrodes to damaged or diseased areas of skin.
- » Coat the loop electrode with electrode gel and place it in the region to be stimulated.
- » Close the loop enough so that the loop electrode is firmly in place, but not so much that it causes any constriction.
- » Clean the loop electrode after each use with lukewarm water and soap.

## 5 Programs


The PhalliMax has three programs that were developed for different areas of application.

Program 1 and 2 are used for the stimulation of the muscles in the cavernous body, whereby program 1 is designed for use with the loop electrode and program 2 is designed for use with thin, self-adhesive electrodes. Program 3 was developed for the stimulation of the pelvic floor muscles.

### 5.1 Program 1

|                  |                  |     |                 |     |
|------------------|------------------|-----|-----------------|-----|
| Frequency        | 30 Hz            |     |                 |     |
| Pulse width      | 90 $\mu$ s       |     |                 |     |
| Modulation times | Ascending phase  | 1 s | Operating phase | 3 s |
|                  | Descending phase | 1 s | Break phase     | 6 s |

- » Coat the loop electrode with electrode gel and place it around the penis, just below the glans. Close the loop enough so that the electrode is firmly in place, but not so much as to cause constriction. Connect one end of the loop electrode to the red cable plug.
- » Place the self-adhesive electrode (50 x 50 mm) above the base of the penis on the lower abdomen. Connect it to the blue cable plug.
- » Set the intensity so that it is very noticeable, but not painful.
- » Apply program 1 for 30 minutes. It should be used in alternation with program 2.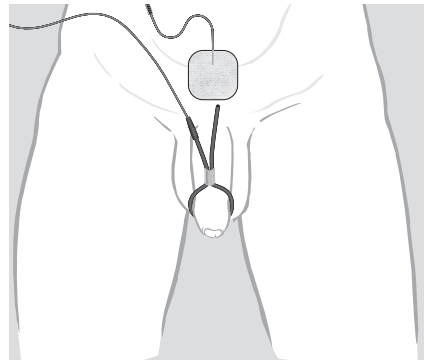


## 5.2 Program 2

|                  |   |
|------------------|---|
| Frequency        | 10 Hz                                       |
| Pulse width      | 150 $\mu$ s                                 |
| Modulation times | Ascending phase 1 s<br>Descending phase 1 s |

|                 |     |
|-----------------|-----|
| Operating phase | 2 s |
| Break phase     | 4 s |

- » Affix the electrodes lengthways on the sides of the penis shaft (cable connection is arbitrary).
- » Set the intensity so that it is very noticeable, but not painful.
- » Apply program 2 for 30 minutes. It should be used in alternation with program 1.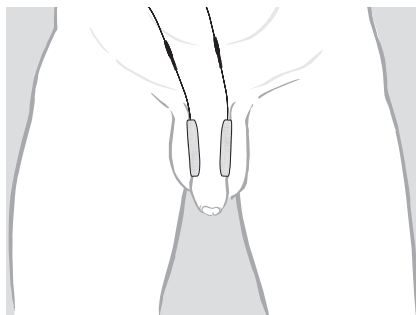


## 5.3 Program 3

|                  |   |
|------------------|---|
| Frequency        | 55 Hz                                       |
| Pulse width      | 250 $\mu$ s                                 |
| Modulation times | Ascending phase 1 s<br>Descending phase 1 s |

|                 |      |
|-----------------|------|
| Operating phase | 5 s  |
| Break phase     | 10 s |

The electrode attached to the blue cable plug is affixed to the perineum for pelvic floor training. The electrode connected to the red end of the cable is positioned above the pubic bone.

- » Set the intensity to trigger noticeable muscle movements in the perineum region.
- » Apply this program for 30 minutes. You can use it in addition to, or in alternation with, program 1 and 2.

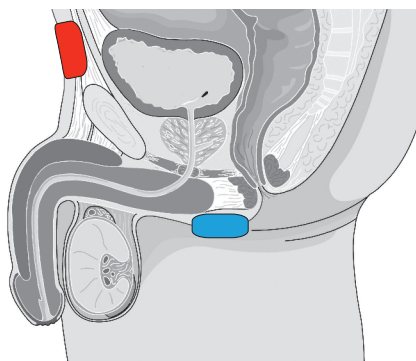## 6 General Information

### 6.1 Warranty

The legal provisions for warranty rights apply as per the German Civil Code.

### 6.2 Guarantee

The manufacturer provides a 12-month guarantee for the PhalliMax device commencing on the date on which the end customer took possession of the device. The guarantee does not apply:

- » to wearing parts and consumables such as electrodes, batteries and connector cables
- » in case of damage incurred as a result of improper use
- » to defects that were previously known by the customer
- » to personal negligence on the part of the customer.

### 6.3 Care and cleaning

No special care products or cleaning agents are required for the PhalliMax. The device and the electrode cables can be cleaned using a soft, lint-free cloth. Please ensure that no moisture enters the device. If moisture does get inside of the device, the PhalliMax must be subjected to a safety inspection prior to any further use. The device can be disinfected using standard disinfection agents, e.g. Meliseptol rapid. Please follow the manufacturer's instructions for use for the disinfection agent you use.

### 6.4 New settings, changes and repairs

The manufacturer is only responsible for the safety and performance of the device if new settings, changes and repairs are performed by the manufacturer or by persons authorized by the manufacturer, and the device is used in compliance with the instructions for use.

### 6.5 Safety checks (§11 MPBetreibV) [Medical Devices Operator Ordinance]

As an operator, you are legally obligated to perform safety checks for this product every 24 months (in accordance with generally accepted technical standards). The safety checks comprise:

1. Checking the accompanying documents to ensure that the instructions for use and the medical device log are present
2. Checking the equipment to ensure it is complete
3. Visual inspection
  - for mechanical damage
  - of all lines and plug connectors for damage
4. Functional safety
  - Checking the output signals at a load resistance of 1 k $\Omega$  (Voltage and current)
  - Checking the output signals at an ANSI load resistance (Voltage and current)
  - Checking the frequency
  - Checking the pulse width

On request, we will be happy to carry out the safety checks for you (fee applies).

## **6.6 Combination with original accessories**

On account of its CE mark classification as a medical device, the PhalliMax can only be used in conjunction with the accessories listed in the instructions for use. Otherwise we cannot guarantee the safety of the user, nor can we honor the terms of the guarantee for the device.

## **6.7 Reference sheet for battery retraction and disposal**

In connection with the sales of products which contain batteries or rechargeable batteries, we are obliged by law (battery law § 18, section 1) to advise you of the following:

The dustbin symbol () marks batteries with hazardous substances as well as the circumstance that batteries must not be disposed of in the household waste but professionally. The chemical term of the hazardous substance is below the dustbin symbol.

You are obliged by law to return used batteries.

You may render used batteries at municipal collecting points or in local trade. We are also obliged to take back used batteries, whereat our obligation restricts to those which are or were part of our product range.

Therefore, you may send back used batteries to us adequately stamped by mail or render these at our warehouse free of charge at the following address:

Pierenkemper GmbH, Dreieiche 7, 35630 Ehringshausen, Germany

Please refer to the following figures to see the respective symbols of batteries with hazardous substances:

Battery contains more than 0.002 % cadmium

Battery contains more than 0.0005 % quicksilver

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